



The Professional's Choice for Dental Products





Since 1864

The Inventors of Hardened Plastic Teeth

With their expertise in shading, blending, curing and carding of teeth, Justi® now manufactures six lines of hardened plastic teeth (Imperial, Imperial ViVO, Justi Blend, Dymon-Hue, Dymon-Hue V, and Triplex), all of which are distributed worldwide. Justi® also formulates and manufactures a wide range of chairside and laboratory products.

OUR PROUD LEGACY



1864

Henry D. Justi receives first US patent, #42491 for tooth molding, and H.D. Justi Co. is formed.




1880

Henry D. Justi builds a tooth factory in Philadelphia, PA.



1892

Henry M. Justi becomes partner. Company name is now H. D. Justi and Son.

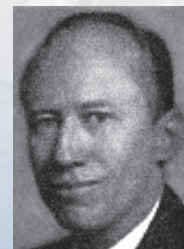

1940

Henry M. Justi Jr. converts factory to plastic tooth manufacturing exclusively and introduces the first plastic tooth line to the world at the ADA Convention, Cleveland, OH.

1961

Henry M. Justi Jr. develops the first Imperial tooth line.

Pozzi®

Since 1971

The Pozzi® Dental Products line is recognized as a world leader in abrasives, stainless steel crowns and bands, laboratory diamonds and specialty items.

1980

Bruno Pozzi

In 1980 the H.D. Justi Company was purchased by a California tooth manufacturing company owned by Bruno Pozzi and other partners. The factory was transferred from Philadelphia to Oxnard, CA, finally merging with American Tooth Industries in 1986.



1986

American Tooth Industries

In 1986 the H.D. Justi Company merges with American Tooth Industries.



**AMERICAN
TOOTH
INDUSTRIES**

American Tooth Industries is comprised of three product brand divisions: Justi®, Pozzi®, and Major®.



Justi® celebrates 159 years in the dental industry with the introduction of a new tooth line, Dymon-Hue V, and tooth line upgrades.

Justi® tooth lines continue to be updated to accommodate all the growing needs of the prosthodontic professional, from quality hardened plastic teeth and crowns, to state-of-the-art tools and equipment.

2017

Emilio Pozzi

All Justi® products continue to be manufactured in the U.S.A. and made with raw materials bought from U.S. factories.



Emilio Pozzi, current President/CEO and son of Bruno Pozzi, the company's founder, continues to move Justi® and Pozzi® companies toward the future. Emilio ensures that we manufacture the best-quality products using updated techniques and new equipment while still maintaining that American-made, personal touch for our products.

Dear Dental Professional,

It is with a sense of great pride that I inform you that the materials used in our Justi® teeth are all produced here in the United States. We buy all our raw materials from U.S. factories. In Oxnard, California, we color and formulate our polymers and monomers, and then we mold the teeth. From beginning to end, quality control is under one roof.

All the products we use to make our teeth are available to you as well, so you can use the teeth and the raw materials with confidence for your needs as a professional. My commitment remains to deliver the best-quality product with competitive pricing now and always.


Sincerely,

Emilio Pozzi, President

"Simple inventions last forever"


Table of Contents

 <p>Since 1864</p>	TEETH	6-15
	TOOTH TRAYS AND CABINETS	16
	ACRYLICS	17-21
	RESINS	22-23
	EQUIPMENT	24-25
	WAX SOLVENT	25


 <p>Since 1971</p>	ARTICULATORS	26
	CROWNS AND PLIERS	27-29
	ABRASIVES	30
	RESINS	31-33
	WIRE AND SOLDER	33-35

Certificates

2023 State of CA
Medical Device Manufacturing License



MDSAP 2025





GROUP/ITEM #	DESCRIPTION
084-50-0610	Justi Imperial ViVO Anterior Card 1x6
084-50-0620	Justi Imperial ViVO Posterior Card 1x8
084-50-0300	Justi Imperial ViVO 1x28

Working Mold Guide

GROUP/ITEM #	DESCRIPTION
084-53-0011	Justi Imperial ViVO WMG Ant/Post (33 Ant. and 24 Post.)

Shade Guide



GROUP/ITEM #	DESCRIPTION
084-51-0429	Justi Imperial ViVO SG 19 Tabs

Justi® Imperial ViVO®

Justi Imperial ViVO® offers an incisal and interproximal blend that give this beautiful line of teeth the unique ability to absorb and reflect surrounding color. This means more latitude for blending with natural teeth—resulting in less resets due to shade, especially in partial denture cases.

Justi Imperial ViVO® offers traditional-quality American molds using the newest *Vita®-type shade system. The effect of this combination is outstanding aesthetics and excellent functionality.

The ability of the teeth to blend into the adjacent dentition has made producing great-looking stay plates much more predictable.

The chameleon-like characteristic of Justi Imperial ViVO® teeth makes shade selection much easier and greatly reduces remakes. The traditional American molds have a very natural look to them.

Imperial ViVO® teeth are suited for:

- Full dentures
- Implant prosthetics
- Telescope and cone constructions
- Attachment dentures
- Model cast prostheses
- Interim solutions

Shades

The Imperial ViVO® line is available in 16 of the Classical *Vita®-type shades, including 3 bleached shades: OM1, OM2, OM3, A1, A2, A3, A3.5, A4, B1, B2, B3, B4**, C1, C2, C3**, C4, D2, D3, D4**.

**These Imperial ViVO® shades can be manufactured upon request. Please inquire for quantity and terms.

Molds


The Anterior mold selection consists of 24 upper and 9 lower molds. Upper: 1G, 1L, 1M, 2S, 3M, 3N, 3S, 5L, 5N, 5S, 7M, 7N, 8M, 8N, 8S, 9G, 10M, 12M, 12N, ΔE12, ΔE22, ΔE32, ΔF42, ΔD55. Lower: U, V, W, X, Y, YS, ΔE, ΔH, ΔN.

The Posterior mold selection consists of 13 upper and 13 lower molds: 34M, 32M, 30M, 33Z, 31Z, 29Z, 34X, 33X, 32X, 33XX, 31XX, 29XX, and new 15° OLU5 & OLL5.

Upper Anterior and Posterior Extra-Large Teeth

Imperial ViVO® Special Upper Anterior AXL Central Incisors and Upper Posterior PXL36 Bicuspid are specialty items that will save you the expense and trouble of making custom teeth when you need to fill a large anterior space in a complete upper or partial denture. The upper posteriors may also be used as molars. These specialty teeth are available in all 19 ViVO® shades.

*Vita® is not a trademark of American Tooth Industries.

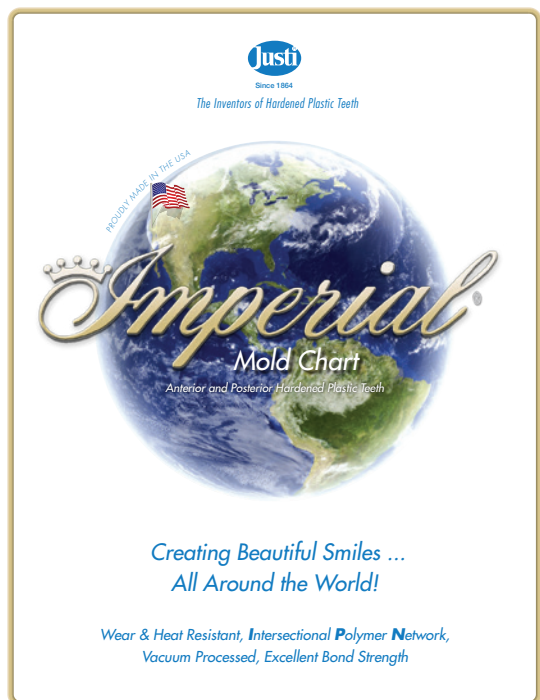

Justi® Imperial® Teeth

The teeth are photogenic, vacuum processed and 100% guaranteed to be bondable to methyl methacrylate denture base materials.

Shades

Justi Imperial anterior and posterior teeth are available in 22 Age Shades®: 17, 18, 19, 20, 21, 25, 35, 45, 50, 59, 60, 62, 64, 65, 66, 67, 68, 69, 70, 77, 81 and 82. Justi Imperial molds include 44 upper anteriors, 13 lower anteriors, 26 upper posteriors and 26 lower posteriors.

Molds

The tooth molds are available in graduated-medium, normal and small sizes. They all have good labial anatomy, and over-polishing is not a major concern. The most sophisticated aesthetics for full dentures can be achieved by using Types 9, 10 and 12 molds. They are specifically designed for minimum difficulty in full denture setup. Type 1 molds are great for partial work because they have a wide neck and almost parallel sides, which provides for adequate gingival coverage.

Sizes

They also feature an extended range of sizes: 3 giant sizes (1G, 5G, 9G) and 3 tiny sizes (1T, 5T, 9T). There are also anatomically shaped lower molds (YL, VL, YS, VS) and very large lowers, such as Mold U.

The Justi Imperial posteriors are available in 33°, 20°, 10° and 0° occlusal forms new 15°, OLU5, OLL5, also in all 22 shades. For the close bite case, which always requires additional grinding, there are three short sizes: 30S, 32S and 34S, which are short on the lingual and long on the buccal for good natural aesthetics, a feature that is virtually exclusive to the Justi Imperial line.

Working Mold Guide

GROUP/ITEM #	DESCRIPTION
084-53-0010	Justi Imperial Working Mold Guide Contains one set of each mold: 49 anteriors, 42 posteriors, all in one shade

- **Cross-linked**
- **Wear Resistant**
- **Solvent Resistant**
- **Fluorescent**



GROUP/ITEM #	DESCRIPTION
083-50-0102	Justi Imperial Anterior Card (1x6)
083-50-0202	Justi Imperial Posterior Card (1x8)
084-50-0301	Justi Imperial Combination (1x28)

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.



Justi® Imperial® Shade Guides

There are 2 different shade guides available for the Justi Imperial tooth line. The Justi Imperial Shade Guide holds 18 shades of this beautiful line, grouped by hue and numbered in accordance with the exclusive Age Shades® System. The Imperial Bleached Shade Guide displays the 4 bleached shades offered in the line. A combo set is available which includes both the 18-shade Imperial Shade Guide and the 4-shade Bleached Shade Guide. Both shade guides are packaged with an optically neutral light blue frame, an Eterna shield and a Kroma Tab, which allows the technician to view teeth on a neutral background.

GROUP/ITEM #	DESCRIPTION
084-51-0425	Imperial Shade Guide w/Kroma Tab: 18 Age Shades®
084-51-0426	Imperial Aged Shades Combo Shade Guide Set. 18 Imperial shades plus 4 Imperial Bleached Shades.
084-51-0425TS	Imperial Bleached Shades 4-Tab Shade Guide



Justi® Imperial® Bleached Shades

Justi Imperial bleached shades 17, 18, 19 and 21 are perfect for a whiter and brighter youthful-looking restoration. Technicians can make a partial that matches bleached natural dentition with ease. Justi Imperial Bleached Shades are available in all Imperial anterior and posterior molds.

GROUP/ITEM #	DESCRIPTION
083-50-0102TS	Imperial Bleached Shades Anterior 1x6
083-50-0202TS	Imperial Bleached Shades Posterior 1x8
084-50-0301TS	Imperial Bleached Shades 1x28



Justi Imperial® Block Teeth Uppers and Lower 1x8

Made from highly cross-linked blended acrylic material, these block teeth are 4 posterior teeth joined together on a flat plane. The zero degree teeth are a time-saving factor in articulation and mounting of the blocks in a denture wax up. They are also available in upper and lower mold 30M only. The cusplless carvings eliminate much of the lateral stress during use.

Shades

Justi Imperial block teeth are available in Imperial shades 20, 25, 45, 59, 62, 65, 66, 67, 69 and 77, and in *Vita®-type shades A1, A2, A3, A3.5, A4, B1, B2, B3, C1, C2, C4, D2, and D3 as well as new bleached shades 17, 18, 19 and 21.

GROUP/ITEM #	DESCRIPTION
084-50-0701UShade #	Justi Imperial Block Teeth-Uppers (1x8)
084-50-0701LSHade #	Justi Imperial Block Teeth-Lowers (1x8)

*Vita® is not a trademark of American Tooth Industries.



Shade Guide



Two new striking bleached shades have been added to the Dymon-Hue Tooth line & Shade Guide: Shades H21 & H22.

GROUP/ITEM #	DESCRIPTION
084-19987	Dymon-Hue HPT SG 15 Shades 12x2

H21, H22, H59, H60, H61, H62, H65, H66, H67, H68, H69, H77, H78, H81, H87

Justi® Dymon-Hue® HPT Hardened Plastic Teeth

Dymon-Hue HPT (hardened plastic teeth) are manufactured with life-like aesthetics that are blended to be similar to those of natural teeth. Dymon-Hue HPT are made of high-quality synthetic resin. They are wear resistant, cross-linked and vacuum processed, and have superior hardness. The cross-linking system used is an interpenetrated, multilayered, intersectional polymer network that is solvent and heat resistant. The pigmented copolymer incorporates inorganic-type pigments that are color stable and unaffected by light or heat. The denture base side of the tooth extending to the neck portion is constructed of a solvent-resistant, softer polymer that assures a 100% chemical bond to all methyl methacrylate denture base acrylics. No diatoric retention is necessary.

GROUP/ITEM #	DESCRIPTION
082-19974	Dymon-Hue Anterior Card 1x6
082-19973	Dymon-Hue Posterior Card 1x8

Molds and Sizes

Dymon-Hue HPT are available in 35 anterior molds and 40 posterior molds, including 0° Planatological™, 10° Symmetry™, 20° Semi-Anatomical, and 33° Anatomical and Short-Bite. A new Extra-Large Anterior Mold, #268, has been added to further expand this line's selection. They are also available in anatomical short-bite molds that provide the technician additional space, eliminating tooth adjustments in many cases.

Justi® Dymon-Hue® HPT Repair Kit


The kits offer a customized assortment of either anterior or posterior teeth, in your choice of 15 shades, and American-style molds ranging from small to extra large.

GROUP/ITEM #	DESCRIPTION
084-19981	Anterior Repair Kit 108 teeth (18-1x6 cards) Uppers and Lower
084-19982	Posterior Repair Kit 128 teeth (16-1x8 cards) Uppers and Lower

Working Mold Guide

GROUP/ITEM #	DESCRIPTION
084-19980	Contains: one set of each mold, in your shade of choice, 35 anteriors and 40 posteriors

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.

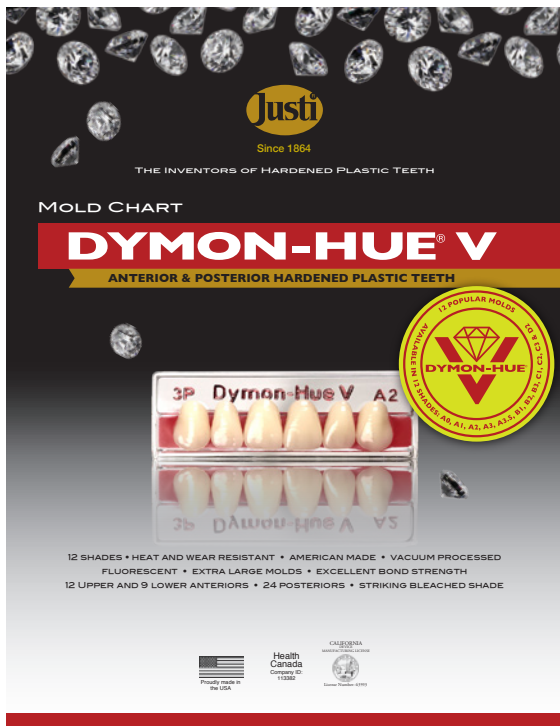

Justi® Dymon-Hue® V

Justi® Dymon-Hue® V – to enhance and extend the Dymon-Hue tooth line, manufactured with lifelike aesthetics that are blended to be similar to those of natural teeth, the Dymon-Hue shades now include 12 popular *Vita® Shades, 12 upper anterior molds, 9 lower anterior molds, and 12 upper and lower posterior molds.

Shades (12)

A0, A1, A2, A3, A3.5, B1, B2, B3, C1, C2, C3, D2

Molds and Sizes

12 Upper Anterior Molds: 3P, 2P, 3N, A25, A26, 266, 267, 264, 2D, 4H, 136, 137

9 Lower Anterior Molds:

3P, 2P, 3N, 26, 2D, 4H, 3R, 46, 47

12 Upper and Lower Posterior Molds:

(20°) 29M, 29L, 31M, 31L, 33M

(Anatomical 33°) 30M, 32M, 34M, 34L

(Planatomical TMO°) 31MF

(Symmetry TM10°) 32F, 33F



Dymon-Hue is available in *Vita® Shades & 12 popular molds

GROUP/ITEM #	DESCRIPTION
085-19973/V	Dymony-Hue V Anterior 1x6
085-19974/V	Dymony-Hue V Posterior 1x8


Dymon-Hue V 12 Tab Shade Guide

Dymon-Hue V tooth line is available in 12 *Vita® Shades: A0, A1, A2, A3, A35, B1, B2, B3, C1, C2, C3, D2

GROUP/ITEM #	DESCRIPTION
085-19975/SG	Dymon-Hue V 12 Tab Shade Guide



*Vita® is not a trademark of American Tooth Industries.

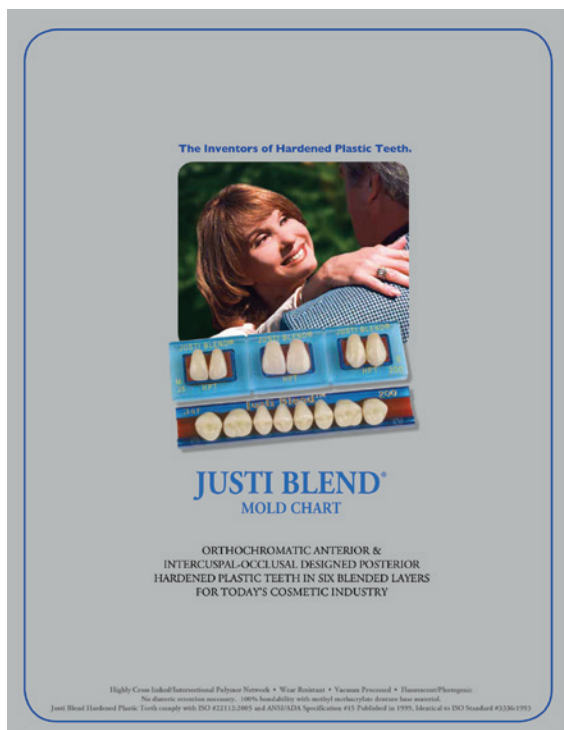

Justi® Blend Teeth Multiblended Hardened Plastic Teeth

Justi Blend brings a fresh and natural look to multiblended characterized plastic anterior teeth. It is the right choice for today's cosmetic industry. Justi Blend is a highly crosslinked, fluorescent, vacuum-processed, laminated plastic tooth. It is unique in that the structure of this tooth line creates the vitality of a natural tooth. The centrals, laterals and cuspids each have a different tone, and this combination of tones blend together in one set. The semitranslucent body helps create a depth within the tooth and reflects color from within, just as dentin does in a natural tooth.

Molds and Sizes

Justi Blend posteriors are designed to articulate with Justi Blend anteriors as well as Dentsply's Bioblend and Bioblend IPN anteriors. Justi Blend teeth are available in 27 upper anterior and 8 lower anterior molds as well as 18 articulating upper and lower posterior molds in 0°, 10°, 20° and 33° anatomical medium and long.



Shade Guides

The Justi Blend Maxillary Centrals Shade Guide displays 9 blended shades and is packaged in an optically neutral light blue frame with an Eterna shield and a Kroma tab that allows the specialist to view the teeth on a neutral background. This shade guide can be customized with the name of a laboratory or clinic plus a marketing message.

The teeth are photogenic, vacuum processed and 100% guaranteed to be bondable to methyl methacrylate denture base materials.

GROUP/ITEM #	DESCRIPTION
079-58-0001	Justi Blend Anterior Teeth (1x6)
079-58-0003	Justi Blend Posterior Teeth (1x8)
084-58-0004	Justi Blend Combination (1x28)

9 shades: 200, 202, 204, 206, 208, 210, 212, 214 and 216.

GROUP/ITEM #	DESCRIPTION
084-59-0026/C	Justi Blend Maxillary Centrals Shade Guide

Working Mold Guides

GROUP/ITEM #	DESCRIPTION
084-59-0011	Justi Blend Posterior Working Mold Guide (contains one set of each mold - 35 sets - all in one shade)
084-59-0010	Justi Blend Anterior Working Mold Guide (contains one set of each mold - 35 sets - all in one shade)

- Cross-linked
- Wear Resistant
- Solvent Resistant
- Fluorescent

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.



SHADE 20		UPPER Δ 30 X
LOWER 20		Δ 30 X
SHADE 20		UPPER Δ 34 X
LOWER 20		Δ 34 X
SHADE 20		Δ 36 X
LOWER 20		Δ 36 X
SHADE 20		Δ 32 X
LOWER 20		Δ 32 X

Delta^Δ™ Anatomic Posterior Teeth

These anatomically pleasing 10° posterior teeth are suitable for complete and removable partial dentures. Their carved cusps are very similar in shape to Dentsply Anatoline® teeth. The tooth aesthetics and functionality are complementary with excellent chewing efficiency. This 10° tooth can be used in lingual occlusion configurations or in applications where gentle excursions can offer the patient proper mastication.

Imperial Delta^Δ™ teeth are available in four sizes, Δ 30X, Δ 32X, Δ 34X and Δ 36X, in all 22 Imperial shades and all 19 Imperial ViVO shades with highly cross-linked intersectional polymer network material with these attributes:

maximum wear resistance, vacuum processed, no diatoric retention is necessary, and 100% bondability with methacrylate denture base material.

GROUP/ITEM #	DESCRIPTION
083-54-0010	Imperial Delta ^Δ ™ Posterior Teeth
083-54-0020	Imperial ViVO® Delta ^Δ ™ Posterior Teeth



Justi® Single Teeth Mix and Match Plastic Teeth

For added savings, laboratories can purchase uncarded single teeth. Minimum quantity is 500 teeth (mix and match per tooth line) per order. Allow ten days' delivery time for orders of single teeth. For special orders, delivery date must be confirmed by a proforma quotation. All purchases are final.

GROUP/ITEM #	DESCRIPTION
081-19973-1	Dymon-Hue HPT Anterior
081-19974-1	Dymon-Hue HPT Posterior
081-50-0104	Justi Imperial Anterior
081-50-0204	Justi Imperial Posterior
081-58-0001/S	Justi Blend Anterior
081-58-0003/S	Justi Blend Posterior

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.



Justi® Imperial® Flatback Plastic Slotted Facings

Justi Imperial Facings are made of the same wear-resistant formula as our Justi Imperial tooth line. The facings are available in a selection of Imperial shades and are pigmented to match the shade guide when used with a standard metallic backing.

The advantages of plastic facings are:

1. Freedom from breakage. They will not chip or fracture while being ground-in and will not break in normal mouth service.
2. Elimination of glazing after grinding to tissue adaptation. Polishing restores the surface so that it is well tolerated by mucosa. Porcelain must be reglazed to prevent irritation.
3. Adaptability. They may be easily ground to any shape, recarved on the labial or characterized. In case of extreme resorption, pink material may be added to the gingival with a pleasing effect.
4. Superior aesthetics. They have distinctive anatomy and fidelity of shade. Justi Imperial Facings may be ground to any shape and can be characterized or recarved on the labial to match adjoining teeth. A preformed plastic backing is available and can be easily modified. Imperial Facings meet or exceed ADA specification #15.

GROUP/ITEM #	DESCRIPTION
084-CL52-0102	Centrals and Laterals (1x4)
084-CUSP520202	Paired Cuspids (1x4)
084-52-0104	Facings Shade Guide (in 8 shades)
084-52-0101	Single Facing (1x1)
077-52-0105	Facings Backings (50 pieces)



GROUP/ITEM #	DESCRIPTION
084-52-0103	Special Facing Assortment Kit
084-52-0104	Facings Shade Guide in 8 Shades



Special Facing Assortment Kit

A working mold guide containing 37 x 4 usable teeth in assorted shades. It has 37 popular molds in 8 popular Imperial shades: 60, 62, 65, 66, 67, 69, 77 and 81. They are wear resistant, stain resistant and designed to fit all standard backings.

Bambino® Teeth

The Bambino tooth is an aesthetic and functional solution to the problem of the empty spaces left by deciduous teeth.

Made of a copolymer cross-linked resin, this tooth line guarantees impermeability to infiltration of fluids and bacteria, thus preventing discoloration and changing of dimensional character. Studied and developed by orthodontic specialists as a simplified alternative to orthodontic appliances, these teeth are ideal for use as a space maintainer to ensure a correct line of teeth where both articulation and aesthetic are concerned. Use of these teeth in articular rehabilitations allows for proper mastication, which can only be obtained with perfect articular balance. Available in a single-mold upper and lower anteriors and upper and lower posteriors. Mounted on dark plastic blocks in 2 shades: 59 (milk white), 61 (yellow-straw white).

GROUP/ITEM #	DESCRIPTION
084-10077	Justi Bambino Deciduous Upper Anterior Card (1x6)
084-10078	Justi Bambino Deciduous Lower Anterior Card (1x6)
084-10177	Justi Bambino Deciduous Upper Posterior Card (1x4)
084-10178	Justi Bambino Deciduous Lower Posterior Card (1x4)
084-10278	Justi Bambino Deciduous Combination Card (1x20)

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.



Lingual Bladed Teeth by Levin

by Bernard Levin, D.D.S., M. Ed.,
Professor Emeritus, USC School of Dentistry

Main Features:

- Provide maximum chewing efficiency plus the aesthetics of natural teeth
- Are articulated to achieve bilateral balance in most clinical situations
- Can be used for lingualized occlusion or monoplane (flat teeth) occlusion




Lingual bladed teeth are manufactured with color-stable, cross-linked acrylic resin and unbreakable surgical stainless steel blades. Crafted with aesthetics in mind, the blades are barely visible. The cross blades that are on the lingual cusps of the maxillary second premolars and molars enable greater masticatory efficiency. Lingual Bladed Teeth are available in Imperial®/Bioform® shades 59, 62, 65, 66, 67 and 81 and bleached 17, 18, 19 and 21 in mold 30M, in 20° and 33 degrees and in mold 28M, in 33 degrees. Also, they are available in *Vita® type shades A1, A2, A3, A3.5, A4, B2 and B3, in mold 30M, in 20° and 33° and in mold 28M, in 33°.



Complete manual available online at
www.americantooth.com

GROUP/ITEM #	DESCRIPTION
083-54-0001/SHADE	Lingual Bladed Teeth 33°M (2x8)
083-54-0002/SHADE	Lingual Bladed Teeth 20°M (2x8)
083-54-0003/SHADE	Lingual Bladed Teeth 33°S (2x8)
077-65-0057	Lingual Bladed Tooth Manual

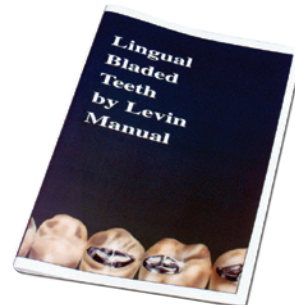

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.



Comes in 22 Imperial® and 7 *Vita® Shades

A1-A2-A3-A3.5-A4-B2-B3

*Vita® Shades is not a trademark of American Tooth Industries.

GROUP/ITEM #	DESCRIPTION
083-54-0005	Mastix Copolymer Bladed Cusps (1x16)

Justi Mastix™ Hardened Copolymer Plastic Teeth

These posterior teeth are made of hard copolymer enamel that will allow perfect mastication by the cutting shape of the maxillary cusps of the second bicuspid and the first and second molars, left and right. The cusps are positioned to allow an effective lingualized occlusion, making this arrangement the most powerful arrangement for the edentulous patient. These teeth allow gentle but powerful mastication performance. This design ensures comfort, functional performance and aesthetic values without undue wear on the opposing plastic teeth.

Sold only in combo kit 1 x 16.

Molds and Shades

Kit includes 1 Imperial Mold, Combo 1 x 16, Upper and Lower.

Available in 1 of 3 sizes: 33° - 30M; 20° - 30M; 33° - 28M

Kits are available in 22 Imperial shades and 7 *Vita® shades:

17, 18, 19, 20, 21, 25, 35, 45, 50, 59, 60, 62, 64, 65, 66, 67, 68, 69, 70, 77, 81 and 82 and A1, A2, A3, A3.5, A4, B2 and B3.



Comes in 22 Imperial® and 7 *Vita® Shades

A1-A2-A3-A3.5-A4-B2-B3

*Vita® Shades is not a trademark of American Tooth Industries.

GROUP/ITEM #	DESCRIPTION
083-54-0008	BlockCutter Copolymer Bar Bladed (1x16)

Justi BlockCutter® Hardened Copolymer Plastic Teeth

These block teeth were designed for very fast setup and feature a revolutionary design allowing dentures to chew more efficiently with less exertion. This revolutionary chewing method is facilitated by mandibular left and right blades made of hard plastic. These plastic blades act like a cutting knife chopping food in the patient's mouth. This design enhances the power of a removable denture to perform mastication activities and delivers performance heretofore unrealized in a denture. All this is done without wearing the opposite maxillary plastic block teeth.

Justi BlockCutters are made of very hard enamel material specially formulated for easy mastication function. Justi BlockCutter teeth are sold in a combo kit 1 x 16 in size 30M only, which accommodates 70% of cases.

Molds and Shades

The BlockCutter comes in one mold size: 0° - 30M.

Kits are available in 22 Imperial shades and 7 *Vita® shades:

17, 18, 19, 20, 21, 25, 35, 45, 50, 59, 60, 62, 64, 65, 66, 67, 68, 69, 70, 77, 81 and 82 and A1, A2, A3, A3.5, A4, B2 and B3.

Justi Hardened Plastic Teeth comply with ANSI/ADA Specification #15, published in 2008, identical to adoption of ISO #22112.



44-Drawer Tooth Counter

Tooth Line	Tooth Capacity	
	Cards per Drawer per Counter	Total Cards
Dymon-Hue HPT Anteriors	72	3,168
Dymon-Hue HPT Posteriors	104	4,576
Jusit Imperial Anteriors	80	3,520
Justi Imperial Posteriors	104	4,576
Justi Blend Anteriors	64	2,816
Justi Blend Posteriors	104	4,576

Counter Dimensions: 19" x 19" x 47 1/4"*

*Allow an additional 2 inches when casters are installed.

Does not include inserts

GROUP/ITEM #	DESCRIPTION
086-55-0200	44-Drawer Tooth Counter
086-55-0101/T8	Tooth Counter Tray 8 Slots



27-Drawer Lab Tooth Counter

Laminated material; 27 drawers with a capacity of 1,512 anterior and/or posterior cards. Can be used for any Justi tooth brand.

Counter Dimensions: 19" x 19" x 33 1/2"*

*Allow an additional 2 inches when casters are installed.

GROUP/ITEM #	DESCRIPTION
086-55-0201	27-Drawer Lab Tooth Counter
086-55-0201/T	Tooth Counter Trays 10 & 4 Slots



Lab Tooth Cabinet

Made of heavy plastic. Ten drawers with 1 tooth insert each. Will hold 320 to 560 anterior and/or posterior tooth cards. Can be used for all Justi tooth brands.

Cabinet Dimensions: 11 1/4" x 13 3/4" x 12 1/2"

GROUP/ITEM #	DESCRIPTION
086-T8030	Justi Lab Tooth Cabinet 3000

Lab Drawer & Inserts

GROUP/ITEM #	DESCRIPTION
077-65-0015	Universal Tray
077-65-0016	Tooth Strips Universal
077-65-0007	Tooth Drawer Insert
077-65-0007/S	Tooth Strip 20-Set Tooth Drawer Insert A/P
077-65-0027	Tooth Drawer Thick Insert for Anteriors
077-65-0027/S	Tooth Strip 36-Set Tooth Drawer Insert Ant.
077-65-0028	Tooth Drawer Thick Insert for Posteriors
077-65-0028/S	Tooth Strip 28-Set Tooth Drawer Insert Post.





Justi® Cold-Cure Ortho Resin

An autopolymerizing material primarily used in the construction of orthodontic appliances. The powder is clear and when used with the transparent liquid, will produce a clear base material. A pink liquid is also available, which when used with the clear powder, will produce a translucent pink base material. The powder is a low-flow polymer that absorbs the monomer rapidly. This prevents the material from slumping when using the liquid dropper technique.

GROUP/ITEM # DESCRIPTION

Justi Ortho Resin Kit - 1 lb. powder, 8 oz. liquid, eye dropper and 3 oz. jet dispenser

072-05-0001	Ortho Resin Kit w/Pink Liquid
072-05-0006	Ortho Resin Kit w/Clear Liquid
072-05-0014	Ortho Resin Kit w/Crystal Liquid

LIQUID REFILLS

072-05-0003	Pink - 8 oz.
072-05-0009	Pink - 1 qt.
072-05-0005	Pink - 1 gal.
072-05-0007	Clear - 8 oz.
072-05-0010	Clear - 1 qt.

LIQUID REFILLS

072-05-0008	Clear - 1 gal.
072-05-0015	Crystal - 8 oz.
072-05-0016	Crystal - 1 qt.
072-05-0017	Crystal - 1 gal.

POWDER REFILLS

072-05-0002	1 lb.
072-05-0004	25 lb.
072-05-002/7	7 lb.



Justi® Denture Base Materials High Impact for Traditional and Miracleflask Microwave Technique

A high-grade methacrylate polymer combined with a cross-linked, solvent-resistant monomer has advantages not found in ordinary denture base materials. It has a short setting and long working time. It is hard, dense and structurally strong, having good model adaptation and high color fidelity. The high-quality cross-linked monomer prevents crazing from solvents such as ether, alcohol, monomer and acetone, etc. This material will bond with any of Justi's tooth lines.

GROUP/ITEM # DESCRIPTION

Justi Denture Base Kit (20 units)* 1 lb. powder and 8 oz. clear liquid

072-04-0001	Regular pink powder
072-04-0006	Dark powder
072-04-0009	B Shade powder
072-04-0012	Regular fibered powder
072-04-0015	Dark fibered powder
072-04-0018	Clear powder
072-04-0033	Ethnic Natural
072-04-0037	Ethnic "A"
072-04-0041	Ethnic "B"
072-04-0045	Regular orig. fibered

Refills - 1 lb. powder

072-04-0002	Regular pink powder
072-04-0007	Dark powder
072-04-0010	B Shade powder
072-04-0013	Regular fibered powder
072-04-0016	Dark fibered powder
072-04-0019	Clear powder
072-04-0034	Ethnic Natural
072-04-0038	Ethnic "A"
072-04-0042	Ethnic "B"
072-04-0046	Regular orig. fibered

GROUP/ITEM # DESCRIPTION

Refills - 7 lb. bulk powder

072-04-0025	Regular pink powder
072-04-0026	Dark powder
072-04-0027	B Shade powder
072-04-0028	Regular fibered powder
072-04-0029	Dark fibered powder
072-04-0030	Clear powder
072-04-0035	Ethnic Natural
072-04-0039	Ethnic "A"
072-04-0043	Ethnic "B"
072-04-0047	Regular orig. fibered

Refills - 25 lb. powder

072-04-0004	Regular pink powder
072-04-0008	Dark powder
072-04-0011	B Shade powder
072-04-0014	Regular fibered powder
072-04-0017	Dark fibered powder
072-04-0020	Clear powder
072-04-0036	Ethnic Natural
072-04-0040	Ethnic "A"
072-04-0044	Ethnic "B"
072-04-0048	Regular orig. fibered

GROUP/ITEM # DESCRIPTION

Refills - Liquid

072-04-0003	MicroLink Liquid 8 oz.
072-04-0031	MicroLink Liquid 1 qt.
072-04-0005	MicroLink Liquid 1 gal.
072-04-0021	Micro-Liquid 8 oz.
072-04-0032	Micro-Liquid 1 qt.
072-04-0022	Micro-Liquid 1 gal.

Shade Guide

072-04-0102	Justi Denture Base Shade Guide - 10 Shades
-------------	--



Justi® Repair Material

A universal-type repair material, applicable for repairing all types of acrylic dentures, temporary prostheses and indirect relines. This self-curing methyl methacrylate material is cross-linked and internally pigmented to give a dense, even texture. Justi Repair Material is available in 3 shades of powder: Regular Pink, Regular Fibred Pink and Clear.

GROUP/ITEM #	DESCRIPTION	GROUP/ITEM #	DESCRIPTION
	Justi Repair Material Kit (20 units) 1 lb. powder and 8 oz. liquid	072-06-0003	Liquid 8 oz.
072-06-0001	Regular pink	072-06-0015	Liquid 1 qt.
072-06-0004	Regular fibred pink	072-06-0013	Liquid 1 gal.
072-06-0006	Clear		Shade Guide
	Refills - 1 lb. powder	072-06-0100	Repair Material Shade Guide
072-06-0002	Regular pink		
072-06-0005	Regular fibred pink		
072-06-0007	Clear		
	Refills - 7 lb. bulk powder		Refills - 25 lb. bulk powder
072-06-0014	Regular pink	072-06-0010	Regular fibred pink
072-06-0016	Regular fibred pink	072-06-0011	Regular pink
072-06-0017	Clear	072-06-0012	Clear



Justi® Tray Material

A self-curing-type acrylic for construction of precision custom trays and stabilized occlusal bite rims. Justi Tray Material is a coarse-type powder that has a high degree of elasticity with a moderate curing time to allow ample working time. After the initial dough stage, the material can be manipulated without sticking to the hands. Having a dead pack, it can be adapted to the model without any significant shrinkage after the cure. The surface roughness aids in the retention of the impression material.

GROUP/ITEM #	DESCRIPTION
072-03-1030	30-Unit Professional Package
072-03-1025	Bulk Shipment Contents: Powder - 1 25 lb. drum, Liquid - 1 gallon
	Refills:
072-03-0499	Powder - 1 25 lb. drum
072-03-0699	Liquid - 1 gallon



IMPAK Transparent

A revolutionary new “soft” acrylic with MEMORY! IMPAK is not just another soft reline material, but has many remarkable uses which traditional soft reline materials cannot match. IMPAK material is used extensively by both professional and amateur athletes for comfortable, precision-fitting mouth protection. IMPAK can also be used to fabricate bruxism appliances with hard biting surfaces, night retainers following orthodontic correction, or as a retainer for surgical packs and splints.

IMPAK Transparent Mouth Guards & Hard Bite Splints Advantages

- All material is clear
- Time-Tested – Over a Million Appliances Fabricated
- Economical Start-Up – No Special Equipment Needed
- Adjustable Softness – Alterable Powder/Liquid Ratio
- Excellent Bond – Adheres Well to Traditional Acrylics
- Reduced Labor – Minimal Adjustment to Fit Master
- Reduced Chair Time – Little or No Chairside Adjustment
- Patient Comfort – Effortless, Pain-Free Insertion
- Everlasting Fit – Soft Retention for Easy Repositioning
- Athlete Safety – Custom Fit & Comfort for Most Activities

GROUP/ITEM #	DESCRIPTION
072-3747	IMPAK Transparent 1LB (454G) POWDER ONLY Pound
072-3305	IMPAK Transparent 6LB (2.72 kg) POWDER ONLY Package
072-3307	IMPAK Transparent 25 LB (11.4 kg) POWDER ONLY Pail
072-3748	IMPAK Transparent 11 OZ (325 CC) LIQUID ONLY Package
072-3306	IMPAK Transparent 1 QT (946 CC) LIQUID ONLY Quart
072-3310	IMPAK Transparent 0.5 GAL (1.98 LTR) LIQUID ONLY 2 Quart
072-3311	IMPAK Fiber Pink 1 LB (454G) POWDER ONLY Pound
072-3312	IMPAK Fiber Pink 6 LBS (2.72 kg) POWDER ONLY Package



Justi® Crown & Bridge Formula

An outstanding cross-linked polymethyl methacrylate resin to formulate crowns, bridges and inlays. It is a heat-cure method for permanent cases and long-term provisionals that is available in 17 different dentin shades: 20, 25, 30, 35, 40, 45, 50, 55, 60, 62, 65, 66, 67, 69, 70, 77 and 81. It also comes in two enamels: light incisal and grey incisal and a finishing transparent shade. Complete technique instructions are available on request.

GROUP/ITEM #	DESCRIPTION	GROUP/ITEM #	DESCRIPTION
070-40-0601	1 oz. "SR" Monomer (Solvent Resistant)	070-40-0404/SHADE	4 oz. Powder - all shades
070-40-0604	4 oz. "SR" Monomer (Solvent Resistant)	070-40-0420/SHADE	1 lb. Powder - all shades
070-40-0632	32 oz. "SR" Monomer (Solvent Resistant)	070-42-0421	Crown and Bridge Formula (SR Shade Selector)
070-40-0699	1 gal. "SR" Monomer (Solvent Resistant)	070-40-0401/T	1 oz. Shade Transparent
070-40-0401/SHADE	1 oz. Powder - all shades	070-40-0404/T	4 oz. Shade Transparent
		070-40-0420/T	1 lb. Shade Transparent



Justi® Imperial Namilon

An internally pigmented, cross-linked methyl methacrylate resin for permanent crown and bridge restorations. It is ideal for quality, high-strength, long-term provisional cases such as implants and full-mouth reconstructions. Namilon is to be used with the crown and bridge investment procedures, i.e., wax-up, investment, boil-out and packing. Because Namilon is internally pigmented, it will not streak, bleed or change color. Namilon features an extended working time, and once polymerized in heat, it cures quickly. Restorations are durable, solvent and wear resistant.

GROUP/ITEM #	DESCRIPTION
070-20-1000	Namilon Kit - Contents: 16 - 1 oz. bottles Namilon Powder in Imperial Shades: 60, 62, 64, 65, 66, 67, 68, 69, 70, 77, 81, 82, Incisal A, Incisal B, Opaque and Transparent, 2 - 1 oz. bottles Liquid Resin, 2 - Mixing Jars, 1 - Measuring Spoon, 1 - Glass Pipette, 1 - Imperial Shade Guide, 1 - Namilon Instructions

Justi® Soft Denture Reliner Semi-Rigid, Soft and Very Soft, Heat Cure

An ethyl methacrylate-type material that can be heat cured traditionally or processed in the microwave oven within minutes when using the Justi Miracleflask system. This pink soft liner blends well with most denture base materials. It has a choice of three textures— semi-rigid, soft and very soft. The material is tissue tolerable and will improve mouth comfort from spiny ridges or other causes of chronic soreness. Doctors, technicians and patients will all be exceptionally pleased with this high-quality and long-lasting material.



GROUP/ITEM #	DESCRIPTION
072-14-0001	Justi Denture Reliner Kit - Semi-rigid
072-14-0002	Justi Denture Reliner Kit - Soft
072-14-0003	Justi Denture Reliner Kit - Very soft
072-14-0004	Justi Denture Reliner - 8 oz. Liquid - Semi-rigid
072-14-0005	Justi Denture Reliner - 8 oz. Liquid - Soft
072-14-0006	Justi Denture Reliner - 8 oz. Liquid - Very Soft
072-14-0007	Justi Denture Reliner - 1 lb. Powder



GROUP/ITEM #	DESCRIPTION
070-28-0001	Justi-Pakto Kit

OPTIONAL ACCESSORY



GROUP/ITEM #	DESCRIPTION
070-28-0020	Pressure Pot - 4 quarts

Justi® Pakto System “The Ceramist’s Friend”

Same technique as is used with porcelains. The Justi Pakto® System is a high-quality resin-to-metal alternative to porcelain.

Justi Pakto is a crown and bridge resin material system designed to save time and labor while offering maximum physical properties to simulate the hardness of natural dentition. Justi Pakto is not a methyl methacrylate, a composite or a porcelain. It is 100% poly glycol dimethacrylate resin and is processed in a dry-heat pressurized chamber.

The areas of application are:

- A) Crown and bridge veneers;
- B) Facings on partial dentures and attachment frames;
- C) Inlays of all types;
- D) Maryland-type bridges; and
- E) Temporary crowns and bridges.

Justi Pakto can be used with any ADA-recommended metal with no color change or fractures. The opaque has superior masking qualities and chemically bonds to the tooth-shaded resin material.

Color verification can be done before the resin is cured since the shade is identical when cured. Justi Pakto is shaded to match the Vita-Lumin® Shade System. (Vita-Lumin is not a registered trademark of American Tooth Industries.)

The same tools used to process porcelain can be used for Justi Pakto. The Justi Pakto Curing System uses a 4-quart, dry-heat pressurized pot.

Replacement parts sold separately.

GROUP/ITEM #	DESCRIPTION
--------------	-------------

Pressure Pot System:

070-28-0020	CONTENTS: 1 - Pressure Pot - 4 quarts, 1 - Curing Vessel, 2 - Security Spring Clips, 1 - Air Supply Hose (3-month warranty on system)
-------------	---

Justi Pakto Kit:

070-28-0001	16 - 1 oz. Bottles Justi Pakto Powder in *Vita® Shades: A1, A2, A3, A4, B2, B3, B4, C1, C2, C3, C4, D2, D3, Body Shades, Transparent Overlay and 2 Bottles Incisal, 8 - 1/4 oz. Bottles Justi Pakto Modifying Stains: white, brown, yellow, gray, blue, pink, orange and dark orange, 2 - 1/4 oz. Bottles Justi Opaquer Powder (preshaded), 2 - 1 oz. Bottles of Justi Pakto Liquid, 1 - 1/4 oz. Retention Beads, 1 - 1/4 oz. Bead Adhesive, 2 - Dappen Dishes, 1 - Glass Pipette, 1 - Plastic Scoop, 1 - Instruction Manual. (Dappen dishes, eye dropper and plastic scoop are not available for re-order) 1 - Justi Pakto Shade Guide - 13 shades, double-sided blue frame
-------------	--

Justi Pakto Refills:

070-28-0002/SHADE	1 oz. Pakto Powders - All Shades (Includes Body Shades, Incisal or Transparent)
070-28-0003/SHADE	4 oz. Pakto Powders - All Shades (Includes Body Shades, Incisal or Transparent)
070-28-0004/SHADE	1/4 oz. Pakto Modifying Stains - All Shades
070-28-0005	1/4 oz. Pakto Opaquer - Preshaded

GROUP/ITEM #	DESCRIPTION
--------------	-------------

Pressure Pot System Replacement Parts:

072-28-0021	Curing Vessel Pot
072-28-0022	Security Spring Clip
072-28-0023	Air Supply Hose
072-28-0026	Gasket for Pakto Pot
072-28-0027	Replacement Handle for Pakto Pot - Top
072-28-0027/B	Replacement Handle for Pakto Pot - Bottom
070-28-0030	Justi Pakto Shade Guide
070-28-0032	Justi Pakto Shade Guide - 13 shades, double-sided blue frame

Justi Pakto Shade Guide:

070-28-0030	Justi Pakto Shade Guide - 13 shades on key ring
070-28-0032	Justi Pakto Shade Guide - 13 shades, double-sided blue frame

Justi Pakto Supplies:

070-28-0006	1 oz. Pakto Liquid
070-28-0007	4 oz. Pakto Liquid
070-28-0008	1/4 oz. Retention Beads
070-28-0009	1/4 oz. Bead Adhesive
070-29-0001	1 oz. Justi Pakto Pure Alloy
070-29-0002	3 oz. Justi Pakto Pure Alloy
070-29-0001K	1 Kilo Justi Pakto Pure Alloy
077-66-0003	Pakto Technique Video

Vita-Lumin is not a registered trademark of American Tooth Industries.



Justi® Opaquer

A resin opaquer for masking metal in veneer crowns with outstanding quality of adhesion. It is also very helpful in holding veneer color. Primarily used with Namilon or Justi Crown and Bridge Formula. However, it can also be used with other polymer-based crown and bridge materials. Brown and pink opaquer are used when experience shows they are needed.

GROUP/ITEM #	DESCRIPTION
070-30-1000	Justi Opaquer Kit Contents: 2 - 1/2 oz. Liquid / 5 - 1/4 oz. Powder in shades white, gray, yellow, brown and pink / 1 - Sable Application Brush / 1 - Glass Pipette / 1 - Polyethylene mixing slab / Instructions for use
070-30-0400/SHADE	Powder Refills - All Shades - 1/4 oz.
070-30-0600	Liquid Refills - 1/2 oz.



Justi® Resin Cement For Permanent Cementation

A unique copolymer offering greater adhesive properties and insolubility. Justi Resin Cement offers a lower film thickness and is radio-opaque.

Features:

- Shows adhesion to tooth structure superior to any other cement tested after 24 hours immersion in water.
- Meets the requirement for film thickness in ADA Specification #8, when used according to instructions.
- Available in 6 shades: Natural Yellow, Natural Gray, White, Pink, Brown and Clear.
- Insoluble in mouth fluids.
- As harmless to the pulp as present resin filling materials.
- Radio-opaque.

GROUP/ITEM #	DESCRIPTION
070-80-1000/SHADE	Justi Resin Cement Kit Contents: 2 - 1/2 oz. Powder; 1 - 1/2 oz. Liquid; 1 - Measuring Spoon, 1 - Eye Dropper and Instructions for Use
070-80-0400/SHADE	Powder Refill (1/2 oz.) and a Measuring Spoon Please specify shade when ordering.
070-80-0600	Liquid Refill (1/2 oz.) and Eye Dropper



Justi® Temporary Crown & Bridge Resin

3 Shades - Light, Medium and Dark

An autopolymerizing self-cure powder and liquid resin for fabricating a crown, bridge or splint. It is very hard, color stable and easy to polish and comes in Light (shade 59), Medium (shade 62), and Dark (shade 69). This material has a low exothermic heat peak and therefore, is tolerable to the oral cavity; it does not burn.

GROUP/ITEM #	DESCRIPTION
072-25-0001	Professional Kit - Boxed, 3 oz. each Light, Medium and Dark powders and 8 oz. liquid
Refills:	
072-25-0002	Light Powder (85 grams) 3 oz.
072-25-0003	Medium Powder (85 grams) 3 oz.
072-25-0004	Dark Powder (85 grams) 3 oz.
072-25-0005	Clear Liquid (240 ml) 8 oz.

Justi® Tooth Shades

Cold Cure Temporary Crown and Bridge Acrylic for Labs is a specially prepared formulation for dental laboratories to construct and build strong temporary crowns and bridges. Cross-linking monomer with high craze resistance and toughness. Kit available in Imperial/Bioform type shades 59, 62, 65, 66, 67, 69 and 81. New Bleached shades 17, 18, 19 and 21 are also available in 1lb. bottles.



GROUP/ITEM #	DESCRIPTION
072-25-0010	Complete Tooth Shade Kit Contents: 7 - 3 oz. Tooth Shade powders, 2 - 8 oz. liquid bottles.
072-25-0011/shade	Imperial Bleached Shade Powder Contents: 1 lb. bottle of tooth shade powder (Be sure to indicate the shade- example 072-25-0011/18.)
Refills available:	
072-25-0008	8 oz. bottle of liquid
072-25-0009/SHADE	3 oz. bottle of tooth shade powder (Include shade number with request.)



Super Vipi STG Microflask

Justi® is proud to introduce the industrial-strength Super Vipi STG Microflask to be used in the microwave irradiation technique for curing acrylic resins in the fabrication of dental prostheses. This revolutionary microwave flask is constructed in 2 sections with 4 stainless steel bolts, sleeves and threads for tightening.

- Uniform curing of acrylic resins
- Rapid curing time (13 minutes)
- Complete curing of acrylic resin with no significant free monomer residual
- Clean removal method without damage to the flask
- Reduced energy costs

GROUP/ITEM #	DESCRIPTION
071-67-0004	Justi Super Vipi STG Microflask



Justi® Pakto Pressure Pot

The Justi Pakto Curing System uses a 4-quart, dry-heat pressurized pot. May be used with or without water.

Replacement parts sold separately.

GROUP/ITEM #	DESCRIPTION
070-28-0020	Pressure Pot - 4 quarts
Pressure Pot System Replacement Parts:	
072-28-0021	Curing Vessel Pot
072-28-0022	Security Spring Clip
072-28-0023	Air Supply Hose
072-28-0026	Gasket for Pakto Pot
072-28-0027	Replacement Handle for Pakto Pot - Top
072-28-0027/B	Replacement Handle for Pakto Pot - Bottom



Justi-Kura Curing Pot

Sturdy, absolutely safe, very practical and has a large 6-quart capacity. It is made entirely of stainless steel, and the parts are available individually. This curing pot was designed to satisfy the needs of the modern technician.

GROUP/ITEM #	DESCRIPTION
070-46-0001	Justi-Kura Curing Pot - Complete w/air hose (3-month warranty)
072-46-0002	Air Intake Valve
072-46-0004	Safety Relief Valve
072-46-0005	Pressure Release Valve
072-46-0006	Air Pressure Gauge
072-46-0007	Small Seal for Pressure Pot Safety Lock

Justi® Hydraulic Press

A compact, easy-to-use, affordable quality unit for trial packing and final packaging procedures.

The suggested pressing technique uses 2 compressing levels; Level 1 – 0.5 ton and Level 2 – 1.0 ton.

The 2 holes at the base of the press can be used to secure the unit to a workbench.

Comes with a 6-month warranty from date of purchase.



GROUP/ITEM #	DESCRIPTION
071-67-0015	Hydraulic Press PM 2000

WAX SOLVENT

Justi® Wax Solvent No. 20

An efficient, safe and pleasant scented solvent used for wax elimination and shining after waxing a prosthetic case. An excellent separator for plaster, stone and any other gypsum product. Justi Wax Solvent #20 can also be used in the following applications:

1. Added to boil tank water.
2. To clean counter tops.
3. To clean tools and equipment.
4. To remove wax from clothing.



GROUP/ITEM #	DESCRIPTION
070-01-0616	1 Pint
070-01-0632	1 Quart
070-01-0699	1 Gallon



Pozzi® Galetti Articulator


Featuring: Mechanical Fixation of Models • Precision Made • Original Type • Patented

The Galetti Articulator is solidly constructed of metal alloy casting. It is easy to use, allowing quick fixation of the model. From chairside to patient, it provides immediate adjustment of the occlusal and correct simulation of the central relation. All anatomical movements are possible. Provides correct occlusion with maximum precision. Has perfect occlusal-articular balance to prevent cuspidal interferences and to obtain a balanced prosthetic appliance. For the dentist, orthodontist and oral surgeon, the Galetti Articulator creates an impressive display that is quick and easy to use for case presentations. It can also be used for study models as well as for many additional applications for partial and full denture work.

Special Features

- Allows easy separation of upper and lower components to form two entirely stable working bases.
- Upper anterior ball-bearing joint allows for free motion of upper appliance to check occlusal-articulator balance.
- Hinge axis with bilateral motion capability for anterior, posterior and lateral movements showing working and balancing movements.
- Adjustable vertical angle pin table to raise or lower the bite.

GROUP/ITEM #	DESCRIPTION
001-19893	Galetti Articulator, Complete
001-20071	Galetti Horizontal Bolt, Lower
001-20072	Galetti Front Model Holding Screw
001-20120	Galetti Articulator Retaining Lever - 1 pair
001-20121	Galetti Articulator Screw #5
001-20122	Galetti Back Model Holding Screw

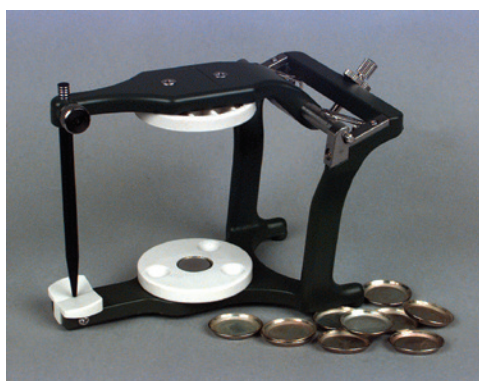

Magnett™ Articulator



This new value-priced addition to the Pozzi Malibu line features four basic movements: mastication, protrusive, right lateral and left lateral. The protrusive movement can be simulated by placing both thumbs on the upper cross member just inside the condular joints and simply pushing toward the back of the condular springs. To achieve right lateral movement, press with the left thumb, and to achieve left lateral movement, press with the right thumb against the cross member.

The Magnett Articulator comes with 10 steel mounting plates.

GROUP/ITEM #	DESCRIPTION
001-20123	Magnett Articulator
001-20124	Mounting Plates Replacement Parts - 10/Pk.

STAINLESS STEEL CROWNS AND PLIERS

This stainless steel crown line is manufactured to meet the needs of all facets of dentistry. The posterior molar crowns are designed with a shallow occlusal anatomy, allowing for more healthy tooth structure after tooth reduction. The shallow occlusal cusps permit complete compatibility with opposing teeth and reduced cuspal interference during normal mastication. The crowns are parallel on the mesial and distal walls. Unlike bell-shaped crowns, these crowns will not rock or cause separation when two or more adjoining crowns are used. We have made these crowns with a thickness that prevents occlusal wear-through that sometimes occurs with the use of thinner crowns. Unlike some of the other crowns made with thinner and less

durable materials, our crowns may be welded too. These are the crowns you want for making space maintainers or Herbst appliances, or for a semipermanent restoration. Through our own research, we have determined these crowns to be of the ideal length: short enough to eliminate excess crown trimming, and long enough to provide optimum length and contour. To make our line complete to the user, we offer first and second primary molars, first and second permanent molars and first and second bicuspids. Pozzi Style 2000 crowns are a match for Unitek® Crowns. The Pozzi® Crown numbers and sizes are the same as Unitek®. Unitek® is not a trademark of American Tooth Industries.



1st Primary Molar

GROUP/ITEM #	DESCRIPTION
040-40-0022	First Primary Molar Crown Kit 72 pcs., 7 sizes
040-40-22(+size)	Reorder pack of 5. Specify crown size and quadrant. *Example: 40-22UL.1

UR	UL
1	1
2	2
3	3
4	4
5	5
6	6
7	7
LR	LL
1	1
2	2
3	3
4	4
5	5
6	6
7	7



2nd Primary Molar

GROUP/ITEM #	DESCRIPTION
040-40-0023	Second Primary Molar Crown Kit 72 pcs., 7 sizes
040-40-22(+size)	Reorder pack of 5. Specify crown size and quadrant. *Example: 40-23UL1

UR	UL
1	1
2	2
3	3
4	4
5	5
6	6
7	7
LR	LL
1	1
2	2
3	3
4	4
5	5
6	6
7	7



1st Permanent Molar

GROUP/ITEM #	DESCRIPTION
040-40-0024	First Permanent Molar Crown Kit 72 pcs., 8 sizes
040-40-24 (+size)	Reorder pack of 5. Specify crown size and quadrant. *Example: 40-23UL1

UR	UL
1	1
2	2
3	3
4	4
5	5
6	6
7	7
LR	LL
1	1
2	2
3	3
4	4
5	5
6	6
7	7
8	8

STAINLESS STEEL CROWNS AND PLIERS



2nd Permanent Molar

GROUP/ITEM #	DESCRIPTION
040-40-0025	Second Permanent Molar Crown Kit - 72 pcs., 8 sizes
040-40-25(+size)	Reorder pack of 5. Specify crown size and quadrant. *Example: 40-25UL-1

UR	UL
1	1
2	2
3	3
4	4
5	5
6	6
7	7
LR	LL
1	1
2	2
3	3
4	4
5	5
6	6
7	7
8	8



1st Bicuspid

GROUP/ITEM #	DESCRIPTION
040-40-0020	First Permanent Bicuspid Crown Kit - 72 pcs., 8 sizes (uppers) 7 sizes (lowers)
040-40-20SIZE*	Reorder pack of 5. Specify crown size and quadrant. *Example: 40-20UL1

UR	UL
00	00
0	0
1	1
2	2
3	3
4	4
5	5
6	6
LR	LL
0	0
1	1
2	2
3	3
4	4
5	5
6	6



2nd Bicuspid

GROUP/ITEM #	DESCRIPTION
040-40-0021	Second Permanent Bicuspid Crown Kit - 72 pcs., 8 sizes (uppers) 7 sizes (lowers)
040-40-21SIZE*	Reorder pack of 5. Specify crown size and quadrant. *Example: 40-21UL1

UR	UL
0	0
1	1
2	2
3	3
4	4
5	5
6	6
7	7
LR	LL
0	0
1	1
2	2
3	3
4	4
5	5
6	6



Small Crown Crimping Plier

This tool is handy for anterior crowns and smaller sizes in the primary molars.

GROUP/ITEM #	DESCRIPTION
040-40-0031	Small Crown Crimping Plier



Regular Crown Crimping Plier

This is the standard in the industry for all posterior crowns.

GROUP/ITEM #	DESCRIPTION
40-40-0030	Regular Crown Crimping Plier



Crown Contouring Plier

For adding contour to the crown for a better fit.

GROUP/ITEM #	DESCRIPTION
040-40-0032	Crown Contouring Plier



Stainless Steel Crown Cutting Tool

Use for trimming anterior crowns.

GROUP/ITEM #	DESCRIPTION
040-40-0028	Small Straight Scissors




Tube Crimping Plier

Used to crimp the tube around a space maintainer wire. It can also be used for placing stops and tubing on arch wires.


GROUP/ITEM #	DESCRIPTION
040-40-0033	Tube Crimping Plier



With the addition of the new Malibu line of abrasives, we continue to provide you with the highest-quality products available in the market today at prices that will fit your budget. Backed by 30 years of experience in the dental abrasives market, the Pozzi Malibu line utilizes the highest grade of materials to meet your highest work requirements. Made with U.S. technology.

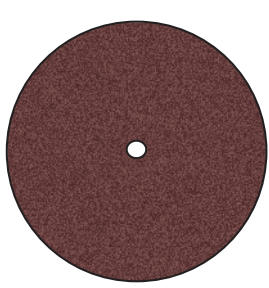

008-39202 PC1

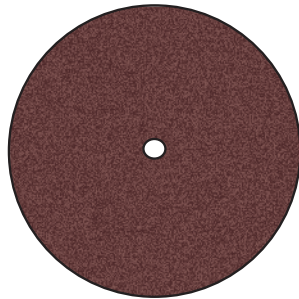


008-39202 PC2

Pin Cutters for Chrome-Cobalt & Non-Precious Alloys

Series PC - Available in 35mm diameter, 0.7mm thickness, and 40mm x 1mm thickness.

GROUP/ITEM #	DESCRIPTION
008-39202 PC1	Malibu Pin Cutters 35mm x 0.7mm 100/pk.
008-39202 PC2	Malibu Pin Cutters 40mm x 1mm 100/pk.

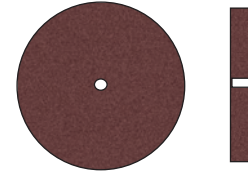


008-39102 AW


Abrasive Wheels for Chrome-Cobalt & Non-Precious Alloys

Series AW - Available in 22mm diameter, 3.5mm thickness.

GROUP/ITEM #	DESCRIPTION
008-39102 AW	Malibu Wheel 22mm x 3.5mm 100/pk.

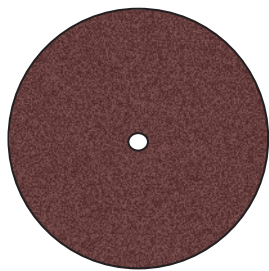

008-39302 RD1

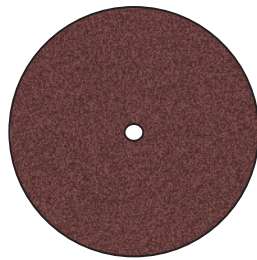

008-39302 RD2


Roughing Discs for Chrome-Cobalt & Non-Precious Alloys

Series RD - Available in 35mm diameter x 1.7 mm thickness and 32mm diameter x 2.2 thickness.


GROUP/ITEM #	DESCRIPTION
008-39302 RD1	Malibu Roughing Discs 35mm x 1.7mm 100/pk.
008-39302 RD2	Malibu Roughing Discs 32mm x 2.2mm 100/pk.



008-39402 S0

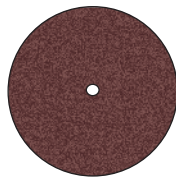

008-39402 S1

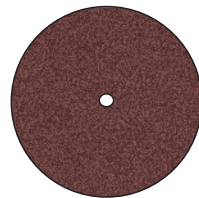

008-39402 S2


Separators for Chrome-Cobalt & Non-Precious Alloys

Series S - Available in 22mm diameter, 0.2 mm and 0.6 mm thickness; 25mm diameter and 0.7 thickness.

GROUP/ITEM #	DESCRIPTION
008-39402 S0	Malibu Separators 22mm x 0.2mm 100/pk.
008-39402 S1	Malibu Separators 22mm x 0.6mm 100/pk.
008-39402 S2	Malibu Separators 25mm x 0.7mm 100/pk. (not pictured)

Mounted Stones for Chrome-Cobalt & Non-Precious Alloys

Series M mounted stones for chrome-cobalt alloy. Available in the shapes below. Blue in color. USA specification. Medium type, cuts fast. Shank 2.35mm diameter. Made of compressed aluminum oxide. Non-contaminating. Guaranteed to cut fast. Recommended speed is between 20,000 and 28,000 rpm.

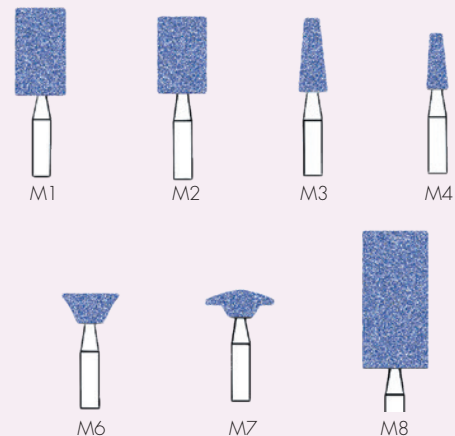

GROUP/ITEM #	DESCRIPTION
008-32084 M1, M2, M3, M4, M6, M7, M8	Series M Mounted Stones for Chrome-Cobalt alloy. Pkg. 100 pcs., Please specify size when ordering M stones.

Pozzi® Ortho Color Liquids

Use of these colors results in beautiful and unique appliances. Colors are dense and bright and are manufactured by our factory in Oxnard, California.



Color Guides

A display of 13 fun colors to assist patients in choosing their color.



GROUP/ITEM #	DESCRIPTION
041-RP6000-00	Color Shade Guide

1 OUNCE BOTTLE	GROUP/ITEM #	DESCRIPTION	
	041-RP6031-01	1 oz. Pozzi Ortho Color Liquid "C"	Pink
	041-RP6122-01	1 oz. Pozzi Ortho Color Liquid "A"	Clear
	041-RP6123-01	1 oz. Pozzi Ortho Color Liquid "B"	Trans. Pink
	041-RP6124-01	1 oz. Pozzi Ortho Color Liquid "D"	Red
	041-RP6125-01	1 oz. Pozzi Ortho Color Liquid "E"	Purple
	041-RP6126-01	1 oz. Pozzi Ortho Color Liquid "F"	Magenta
	041-RP6127-01	1 oz. Pozzi Ortho Color Liquid "G"	Orange
	041-RP6128-01	1 oz. Pozzi Ortho Color Liquid "H"	Fluor. Yellow
	041-RP6129-01	1 oz. Pozzi Ortho Color Liquid "I"	Green
	041-RP6130-01	1 oz. Pozzi Ortho Color Liquid "L"	Turquoise
	041-RP6131-01	1 oz. Pozzi Ortho Color Liquid "M"	Blue
	041-RP6132-01	1 oz. Pozzi Ortho Color Liquid	Black
041-RP6133-01	1 oz. Pozzi Ortho Color Liquid	Gold	

4 OUNCE BOTTLE	GROUP/ITEM #	DESCRIPTION	
	041-RP6031-04	4 oz. Pozzi Ortho Color Liquid "C"	Pink
	041-RP6122-04	4 oz. Pozzi Ortho Color Liquid "A"	Clear
	041-RP6123-04	4 oz. Pozzi Ortho Color Liquid "B"	Trans. Pink
	041-RP6134-04	4 oz. Pozzi Ortho Color Liquid "D"	Red
	041-RP6135-04	4 oz. Pozzi Ortho Color Liquid "E"	Purple
	041-RP6136-04	4 oz. Pozzi Ortho Color Liquid "F"	Magenta
	041-RP6137-04	4 oz. Pozzi Ortho Color Liquid "G"	Orange
	041-RP6138-04	4 oz. Pozzi Ortho Color Liquid "H"	Fluor. Yellow
	041-RP6139-04	4 oz. Pozzi Ortho Color Liquid "I"	Green
	041-RP6130-04	4 oz. Pozzi Ortho Color Liquid "L"	Turquoise
	041-RP6141-04	4 oz. Pozzi Ortho Color Liquid "M"	Blue
	041-RP6142-04	4 oz. Pozzi Ortho Color Liquid	Black
041-RP6143-04	4 oz. Pozzi Ortho Color Liquid	Gold	

8 OUNCE BOTTLE	GROUP/ITEM #	DESCRIPTION	
	041-RP6031-08	8 oz. Pozzi Ortho Color Liquid "C"	Pink
	041-RP6122-08	8 oz. Pozzi Ortho Color Liquid "A"	Clear
	041-RP6123-08	8 oz. Pozzi Ortho Color Liquid "B"	Trans. Pink
	041-RP6124-08	8 oz. Pozzi Ortho Color Liquid "D"	Red
	041-RP6125-08	8 oz. Pozzi Ortho Color Liquid "E"	Purple
	041-RP6126-08	8 oz. Pozzi Ortho Color Liquid "F"	Magenta
	041-RP6127-08	8 oz. Pozzi Ortho Color Liquid "G"	Orange
	041-RP6128-08	8 oz. Pozzi Ortho Color Liquid "H"	Fluor. Yellow
	041-RP6129-08	8 oz. Pozzi Ortho Color Liquid "I"	Green
	041-RP6130-08	8 oz. Pozzi Ortho Color Liquid "L"	Turquoise
	041-RP6131-08	8 oz. Pozzi Ortho Color Liquid "M"	Blue
	041-RP6132-08	8 oz. Pozzi Ortho Color Liquid	Black
041-RP6133-08	8 oz. Pozzi Ortho Color Liquid	Gold	

1 PINT BOTTLE	GROUP/ITEM #	DESCRIPTION	
	041-RP6031-16	1 pint Pozzi Ortho Color Liquid "C"	Pink
	041-RP6122-16	1 pint Pozzi Ortho Color Liquid "A"	Clear
	041-RP6123-16	1 pint Pozzi Ortho Color Liquid "B"	Trans. Pink
	041-RP6134-16	1 pint Pozzi Ortho Color Liquid "D"	Red
	041-RP6135-16	1 pint Pozzi Ortho Color Liquid "E"	Purple
	041-RP6136-16	1 pint Pozzi Ortho Color Liquid "F"	Magenta
	041-RP6137-16	1 pint Pozzi Ortho Color Liquid "G"	Orange
	041-RP6138-16	1 pint Pozzi Ortho Color Liquid "H"	Fluor. Yellow
	041-RP6139-16	1 pint Pozzi Ortho Color Liquid "I"	Green
	041-RP6130-16	1 pint Pozzi Ortho Color Liquid "L"	Turquoise
	041-RP6141-16	1 pint Pozzi Ortho Color Liquid "M"	Blue
	041-RP6142-16	1 pint Pozzi Ortho Color Liquid	Black
041-RP6143-16	1 pint Pozzi Ortho Color Liquid	Gold	



Pozzi® Spruzzo™

Colored Acrylic Powders - Solid Colors

Pozzi blended colored powders were designed for use with our clear monomer. These colors were created for those wanting a more solid color appearance. Designed for use with the salt-and-pepper technique, these colors are sure to catch every teenager's eye! Each color comes in a 4-ounce easy-to-use spray bottle and is available in many popular colors.

	GROUP/ITEM #	DESCRIPTION	
4 OUNCE BOTTLE	040-40-0034	Spruzzo Powder 4 oz.	Yellow
	040-40-0035	Spruzzo Powder 4 oz.	White
	040-40-0036	Spruzzo Powder 4 oz.	Orange
	040-40-0037	Spruzzo Powder 4 oz.	Black
	040-40-0038	Spruzzo Powder 4 oz.	Brown
	040-40-0039	Spruzzo Powder 4 oz.	Blue
	040-40-0040	Spruzzo Powder 4 oz.	Red
	040-40-0040/SG	Spruzzo Shade Guide	



Pozzi® Spruzzo™

Colored Acrylic Powders - Glitter Colors

- Acrylic resins can be molded in complex forms with the use of pressure heat.
- They are translucent and biocompatible.
- They do not lose their color or get pigmented over time, are dimensionally stable.
- They are easy to manipulate.
- Available in yellow, green, clear, and hot pink.

	GROUP/ITEM #	DESCRIPTION	
4 OUNCE BOTTLE	040-40-0050	Spruzzo Powder 4 oz.	Yellow Glitter
	040-40-0051	Spruzzo Powder 4 oz.	Green Glitter
	040-40-0052	Spruzzo Powder 4 oz.	Clear Glitter
	040-40-0053	Spruzzo Powder 4 oz.	Hot Pink Glitter



Pozzi® Ortho Resin

Pozzi Ortho Resin is a self-curing, high-impact acrylic that was developed for orthodontic use in making strong and attractive appliances and retainers. The powder is a "Lo-Flo" formulation that will rapidly absorb the monomer and will not slump, flow or run. Plastic thickness and form can be rapidly accomplished because when Pozzi Ortho powder and liquid are sprayed on the stone cast, the acrylic will stay put and even. Pozzi Ortho Resin Kit is supplied with clear powder and pink or clear liquid. Kit contains: 16 oz. powder, 8 oz. pink or clear liquid, a dropper, a 3oz. jet dispenser and an instruction sheet.

GROUP/ITEM #	DESCRIPTION
033-19822/P	Pozzi Ortho Acrylic 1 lb. kit, pink liquid
033-19822/c	Pozzi Ortho Acrylic 1 lb. kit, clear liquid
033-19823/P	Pozzi Ortho Acrylic 25 lb. kit, pink liquid
033-19823/C	Pozzi Ortho Acrylic 25 lb. kit, clear liquid
033-20024/P	Pozzi Ortho Liquid, 8 oz. pink
033-20024/C	Pozzi Ortho Liquid, 8 oz. clear
033-20032	Pozzi Ortho Powder, 16 oz.
033-20033	Pozzi Ortho Powder, 25 lb.
033-20034/P	Pozzi Ortho Liquid, 1 gal. pink
033-20034/C	Pozzi Ortho Liquid, 1 gal. clear

WIRE AND SOLDER



Premium Ticonium Wire

Premium quality, round clasp wire specially suited for casting with PREMIUM 100 alloy.

12" length (30cm) each wire. Comes 5 Sticks in a tube.

GROUP/ITEM #	DESCRIPTION
015-61622	Premium Ticonium Wire 18-gauge

Pozzi® Retainer Wire (Medical Specification)

- Diamond drawn: yields a perfectly round wire.
- Medium hard temper: a true 1/2 hard wire.
- Easily formed: ideal for hand forming.
- Non-brittle: can be bent 12-14 times before breaking to prevent remakes.




GROUP/ITEM #	DESCRIPTION
015-19907-1	Retainer Wire Straight lengths of 14", tube .025 10 pcs
015-19907-5	Retainer Wire Straight lengths of 14", tube .028 10 pcs.
015-19907-2	Retainer Wire Straight lengths of 14", tube .030 10 pcs.
015-19907-3	Retainer Wire Straight lengths of 14", tube .032 10 pcs.
015-19907-4	Retainer Wire Straight lengths of 14", tube .036 10 pcs.

Gauge Size	mm.	inch	ozs.	# ft.	Group/Item #
22/23	0.6	.024	2/3	22	015-410/6
21	0.7	.028	3/4	22	015-410/7
20	0.8	.032	1	22	015-410/8
19	0.9	.036	1 1/3	22	015-410/9
18	1.0	.040	3/4	11	015-410/1.0
17	1.1	.044	1	11	015-410/1.1
16	1.2	.048	1 1/4	11	015-410/1.2
15	1.3	.052	1 1/3	11	015-410/1.3

Cobalt Clasp Crozat Wire


A wire designed specifically for the Crozat method. It comes in various gauge sizes ranging from 15 to 23. The wire is prepared on a coiled spool in a length of approximately 22 feet. The wire can be soldered in an open flame. Made with a cobalt alloy specifically for surgical and orthodontic appliances.

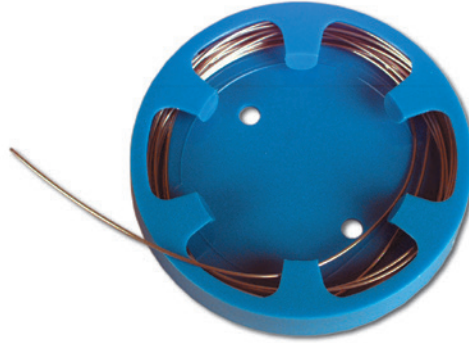

Gauge Size	mm.	inch	ozs.	# ft.	Group/Item #
21	0.7	.028	3	90	015-400/7
20	0.8	.032	3	68	015-400/8
19	0.9	.036	3	54	015-400/9
18	1.0	.040	3	44	015-400/1.0
17	1.1	.044	3	36	015-400/1.1
16	1.2	.048	3	30	015-400/1.2
15	1.3	.052	3	26	015-400/1.3

Standard Round Wire

Each spool is approximately 3 ounces. Chrome-cobalt stainless steel alloy. Important: We recommend using round wire in spools versus straight lengths because the latter has been drawn (processed) one more time, causing a loss of tensile strength. Suitable for all orthodontic and laboratory needs.

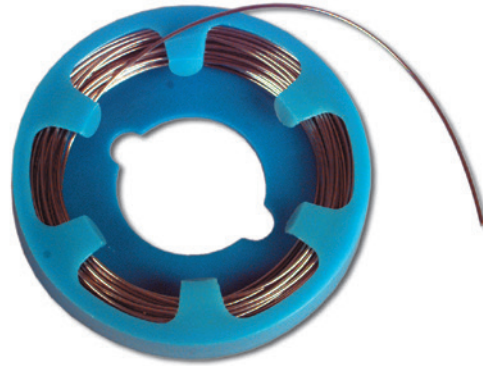

Pozzi® Malibu™ Flux

An all-purpose, low-temperature flux for use in silver brazing. Use with most ferrous and non-ferrous metals, not recommended on aluminum, magnesium and titanium. The active temperature range is 1050°F/566°C.

GROUP/ITEM #	DESCRIPTION
031-32.50.100	Pozzi Malibu Flux 6.5 oz.



THE HISTORY OF POZZI ORTHODONTIC SILVER SOLDER



The Pozzi family has been manufacturing silver solders since 1934. From the melting of different alloys, including 65% of the purest silver, up to the making of rough bulk ingots, the quality control has been consistent, and the resulting product is remarkable. Of course, our solder does not have any cadmium or any other impurities. The rolling of the rough ingots to obtain square and round wires takes at least fifteen steps. The lamination is done with the most precise

gauged rolling mill in the industry. The high content of silver in our orthodontic solder means that very little material is needed in the soldering process, and the possibility of oxidation is nearly eliminated. Solders with a low silver content tend to oxidize heavily due to impurities. The combination of the ingredients in our silver solder and the high quality flux make this product the very best available in the world market.

Pozzi® Malibu™ Orthodontic Laboratory Silver and Brazing Solder

A cadmium-free alloy with free-flowing and ductile characteristics. Easy to use. Manufactured in the U.S.A. for the Pozzi brand. Available in 50 oz., 5 oz. and 1 oz. spools.

Silver Solder
CADMIUM & NICKEL FREE

GROUP/ITEM #	DESCRIPTION
031-32.50.001	1 oz. Spool
031-32.50.005	5 oz. Spool
031-32.50.0500	50oz Spool



- 1 Good strength and ductility on nickel alloys to help avoid stress corrosion cracking.
- 2 Good color match on stainless steel.
- 3 Excellent fluidity that promotes penetration for good strength and elongation.

Diameter	Melting Point	Silver Content
1/32" (.79 mm)	1145° F 618° C	56%



Pozzi

The Professional's Choice For Prosthetic Dentistry

HARDENED PLASTIC TEETH,
ABRASIVES, STAINLESS STEEL CROWNS
AND SPECIALTY ITEMS

Worldwide Catalog Edition



**AMERICAN
TOOTH INDUSTRIES**

1200 Stellar Drive | Oxnard, CA 93033-2402 | 805-487-9868
www.AmericanTooth.com | e-mail: info@AmericanTooth.com